


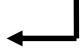

**HASTINGS**  
air energy control, inc.  
*creating a cleaner workplace*

## ZIMEK<sup>®</sup> Treatment Protocol For Vehicles

### Quick Reference Guide

- Plug unit in, use extension cord, if necessary
- Turn power knob to on  $\text{0} \text{⊕} \text{1}$
- Touch screen
- Touch "\*\*\*\*\*"
- Key in pass code "1" – Touch 
- Touch:
- If greater than 75° go to next step. If less than 75°, run cold mode (press the penguin and follow instructions – this takes 40 seconds)
- Touch the steering wheel
- Inspect liquid level. If greater than 4 gallons, proceed. If less than 4 gallons, add one case (empty **all** four gallons from a case into the machine)
- CLEAN VEHICLE PRIOR TO DISINFECTING TREATMENT
- Touch the calculator
- Touch the vehicle that looks similar to the vehicle you intend to treat
- Touch "intensive treatment"  
(Touch "periodic" only if you are sure this is a maintenance procedure and that no infectious contaminants exist)
- Touch "Begin"
- Enter the machine location touch "press here"
- Enter machine location – first two digits represent the department, the next digits represent the vehicle being treated (example – "LF123" represents Lansing Fire Department, vehicle #123, as labeled on the side of the vehicle) touch the "enter" emblem
- Touch "Yes" to confirm location
- Place vehicle decontamination overlay (VDO) on Zimek machine in top opening, place (VDO) in rear bay window or side door window – ensure free air flow into vehicle and tight seal onto vehicle
- Touch "Start", machine will give a 40 second countdown
- Machine will countdown treatment, dwell time then signal complete
- Touch "OK"
- Remove vehicle overlay and place in bag and attach to machine, replace tank cover over reservoir, unplug machine (unless draining required)
- If draining of liquid is necessary to move the machine, Key pass code, touch drain/spray picture at bottom middle, touch "on, use drain attachment. After complete, turn power knob to off  $\text{0} \text{⊕} \text{1}$  Unplug.

